MediSpend Orange

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EN Instructions for use

MediSpend Orange - A lightly viscous liquid with orange flavor that eases the oral administration of tablets, capsules and other solid dosage forms.

What is MediSpend Orange?

MediSpend Orange is uniquely developed to ease the swallowing of solid dosage forms such as tablets and capsules. MediSpend Orange has a pleasant orange flavor that helps mask medicine taste and has a lightly viscous consistency. MediSpend Orange can be used to both aid the administration of intact tablets/capsules or crushed tablets and opened capsules. Consult your health professional and read the patient information leaflet of your medication before doing so.

MediSpend Orange is a Class I Medical Device.

MediSpend Orange is formulated using ingredients that are commonly used in food to make it suitable for all patients.

Intended use

MediSpend Orange can be used to aid with oral intake of different solid dosage forms. These include, but are not limited to:

- Medical pills, tablets, and (liquid) capsules
- Vitamin preparations
- Food supplements
- Other solid dosage forms meant to be swallowed

Caution

- MediSpend Orange is suitable for everyone from the age of 2 who can swallow independently.
- Do not use in case of severe swallowing problems.
- It is not recommended that you crush coated, rapid-dissolving, or sustained release medication.
- Do not use MediSpend Orange with effervescent tablets.
- Do not use in case of known hypersensitivity to any of the ingredients.
- Avoid contact with the eyes.
- If side effects occur, discontinue the use of the product immediately and consult a health professional.
- Keep out of reach of children.

User instructions

- Place the medication on a tablespoon, crush the tablet or open the capsule only if necessary.*
 - Add 10 ml of MediSpend Orange onto the spoon to cover the medication. A dosing cup is included with each MediSpend Orange bottle for optimal dosing.
 - Take the medication and finish by drinking water.**
- When crushing tablets or opening capsules it is recommended to carefully read the patient information leaflet of your medication and/or consult your health professional.
 - ** Finish by drinking water if this is recommended in the patient information leaflet of your medication.

A dosing cup is included with each MediSpend Orange bottle for optimal dosing.

Storage conditions

- Store between 2 and 25 °C.
- Do not use MediSpend Orange more than 12 months after opening and do not use after the indicated expiration date (EXP).

Ingredients

Purified water, modified food starch, natural orange flavor, sodium citrate, citric acid, sucralose, sodium benzoate (<0.1%), malic acid and simethicone.

Content

250 mL

Further information

For more information, please consult your health professional or visit fagron.com.

 Fagron BV, Lichtenauerlaan 182, 3062 ME Rotterdam, The Netherlands

Distributor in Belgium and Luxemburg
Fagron Belgium NV, Venecoweg 20A, 9810 Nazareth, Belgium
fagron.be

This leaflet was last revised in 05/2017.